



# Endo Sealer Solvent™ (Patent Pending)

VISTA APEX

**REF 317033 - 8oz (240mL) Bottle**

## Endo Sealer Solvent™ Indications:

Endo Sealer Solvent™, a root canal cleanser, is intended to irrigate, cleanse, and debride root canal systems including the removal of foreign material and debris during root canal therapy.

## Instructions for Use:

1. Begin the procedure by creating access to the root canal, remove gutta percha (if a retreatment), and establish working length.
2. Fill a clean syringe with Endo Sealer Solvent™ solution.
3. Attach an irrigation needle tip securely to syringe (e.g. 30ga Vista-Probe™ or Appli-Vac™ irrigating tip).
4. Clean the canal with Endo Sealer Solvent™ alternating between flushing and soaking periods of 30 seconds. Continue irrigation until evacuated fluid is clear, it is recommended to use 6-12mL per canal.
5. Irrigate with Chlor-XTRA™, SmearOFF™, and Chlor-XTRA™ again for optimal canal cleanliness.
6. All steps can be activated using the EndoUltra® Cordless Ultrasonic Activator.
7. Evacuate the canal with a micro suction tip (e.g. Elasti-Vac™ Tip).
8. Blot out the canal using 1-2 paper points.
9. Seal and obturate as normal.

## Precautions:

Read SDS for material handling and safety information. Endo Sealer Solvent™ is not intended to remove gutta percha, resin-based sealers, or zinc oxide eugenol sealers. Do not use this product on patients with a known sensitivity to its ingredients. Proper treatment planning and establishment of working length is required prior to use of this product. Following standard of care, isolate the tooth prior to use of this product to prevent unintended tissue contact or accidental ingestion. Properly evacuate the canal during use of this product to prevent unintended tissue contact or accidental ingestion. It is recommended to use irrigation tips with lateral venting (27ga or smaller) to ensure effective distribution of irrigation solution throughout the root canal and minimize the risk of apical extrusion.



**WARNING:** May be harmful if swallowed. Causes mild skin irritation. Causes eye irritation. Wash hands thoroughly after handling. Wear protective gloves/eye glasses. Do not follow Endo Sealer Solvent™ rinse with sodium hypochlorite or vice versa, as this can lead to the production of chlorine gas. Always rinse with water between Endo Sealer Solvent™ and sodium hypochlorite irrigation. Do NOT bind the tip in the root canal. NEVER apply excessive pressure when irrigating this product or any endodontic irrigant. If apical extrusion of irrigant has occurred, seek medical attention immediately. Follow all government regulations for waste disposal. Prevent release to the environment if possible. **FIRST AID: IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. **IF ON SKIN:** Wash off with plenty of soap and water.



**Inter-Med, Inc.**  
2200 South St.  
Racine, WI 53404 USA  
Tel: +1-262-636-9755  
www.vistaapex.com



2°C  
36°F



30°C  
86°F



**Rx ONLY**

Made in USA 317033-I-V-ENG (2)