

Endocator®

QUICK START GUIDE

Endocator is an adjunctive device, it is not a diagnostic device. Regardless of Endocator results, use your clinical judgment to determine the best course of action for treatment.



Refer to User's Manual for detailed instructions.

INSTRUCTIONS FOR SAMPLE COLLECTION

1. Turn on the Endocator.
2. Aspirate canals to remove irrigants.
3. Fill new syringe with approximately 1 mL of sterile water, attach tip.
4. Rinse canals to remove trace irrigants. Leave canals and half the chamber filled with water.
5. Pump tip up and down in each canal 2-3 times to agitate water facilitating sample collection.
6. Aspirate sample into empty syringe, taking all fluid from chamber and half of every canal. [See Figure A].

Note: Take care not to touch irrigation tip prior to sample collection.

