

Irrigation. Simplified.




Chlor-XTRA™ is a 6% sodium hypochlorite that has been enhanced with surface modifiers and wetting agents resulting in lower surface tension improving penetration into lateral canals and isthmuses faster.

Luer-Lock Cap
to minimize waste
and mess

Color coded
with corresponding
syringes* for safety
and convenience

*syringes sold separately

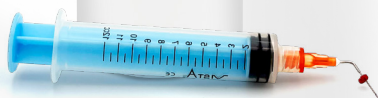

SmearOFF™ is an EDTA-based formula enhanced with Chlorhexidine to combine the benefits of smear layer removal with long lasting cleansing. The combination of EDTA and Chlorhexidine in SmearOFF effectively removes inorganic debris more safely.



SCAN NOW
to download
PDF for easy
implementation!

VistaApex
SOLUTIONS
Value. Added.

The time-saving duo

The combination of Chlor-XTRA™ and SmearOFF™ is a safe, simple and effective irrigation system. These proprietary formulations have been specifically engineered for safety, consistency and reliability. When used together no additional rinsing is required, giving you a true 2-step solution that saves time.

Chlor-XTRA™ 6% Sodium Hypochlorite



Key benefits:

- **Improved contact** – Lower surface tension results in enhanced tissue dissolution
- **High pH** – Alkalinity promotes dissolution of organic debris

Available in 16oz bottle (SKU 503800),
3mL pre-filled syringe (SKU 503825) and
12mL pre-filled syringe (SKU 503850)

SmearOFF™ EDTA/ Chlorhexidine mix



Key benefits:

- **Superior effect** – Removes smear layer in the coronal, middle and apical thirds of the canal¹
- **Safer results** – No precipitate formation when mixed with sodium hypochlorite irrigants, including Chlor-XTRA
- **Highly effective** – Superior chelation and enhanced cleansing compared to standard EDTA irrigants

Available in 16oz (480mL) bottle (SKU 317111)
or 4oz (120mL) bottle (SKU 317112)

Advanced Irrigation Protocol

Save steps and valuable chair time by using our premium irrigants.



STEP 1: Irrigate

During instrumentation and the initial irrigation, use VistaApex Chlor-XTRA, a 6% Sodium Hypochlorite solution with surface modifiers and wetting agents.



STEP 3: Rinse

Rinse with sterile water or alcohol before obturation.



STEP 2: Clean

Remove the smear layer and ensure pristine cleanliness with VistaApex SmearOFF 2-in-1 EDTA + CHX irrigant. A rinse step is NOT required prior to delivering SmearOFF.



One tip for all solutions

Use the VIP Tip 17mm (SKU 315729) or 27mm (SKU 315829) during any irrigation step.

VistaApex Solutions

2200 South Street
Racine, WI 53404

Phone: (877) 857-8213

Email: info@vista-dental.com

Website: <https://vistaapex.com>

References:

1. Stojicic S. et al. (2010). J Endod. 36(9): 1558-62.