



EndoUltra® Activator Tips


VISTA APEX

REF 407706 - Activator Tip 3pk


General Information:

- EndoUltra® Activator Tips resonate down entire length of tip
- Depth markers at 16mm, 17mm and 18mm
- Tip frequency: 40-50kHz (40,000-50,000 cycles/sec)

Installation Instructions:

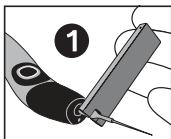


1. Hand thread tip onto handpiece.

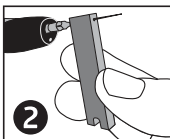

2. Tighten with wrench until tip no longer spins.

NOTE: Do not over-tighten.

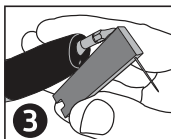

3. Slide tip through wrench hole until it stops.

Bend tip to desired angle
(70° - 90° for best results)

Maintenance & Care:

- Activator Tips must be cleaned and sterilized prior to use.
- Follow detailed reprocessing instructions contained within the IFU.

NOTE: Tips consist of titanium.



Inter-Med, Inc.
2200 South St.
Racine, WI 53404 USA
Tel: +1-262-636-9755
www.vistaapex.com

RxOnly — For dental use only.

CAUTION: U.S. Federal law restricts this device to sale by or on the order of a dental professional.

407703-I-V-ENG (10)